



Drying Instructions

Sterisystem™ Closed Stainless Steel Case Carts



Important – Please pass these instructions on to the people responsible for the cleaning of the Sterisystem™ Closed Stainless Steel Case Carts

For the optimum performance of your case cart make sure to follow the drying instructions below:

- Drying time: 8 minutes at 185°F in **dry air**.
- **Dry air** = air with a relative humidity of maximum 60% at an ambient temperature of 70°F. Converted this is a relative humidity of maximum 1,1% at 185° F.

For additional information or questions about your Sterisystem™ Closed Stainless Steel Case Cart please contact your local Account Manager or Belintra at (864) 642-0227.